

# Retainer Brite<sup>®</sup> is Easy-To-Use

Use daily for best results



- 1** Drop one tablet into enough warm (not hot!) water to cover appliance.



- 2** Place appliance into bubbling blue solution.



- 3** When blue clears (15-20 minutes\*) appliance is clean and odor free!



- 4** Rinse appliance and hands thoroughly with running water.



Keeps  
Appliances and  
Breath Minty-Fresh

**Maintain a professional looking clean between regular dental visits**

\*Wire appliances: DO NOT allow wire appliances to soak for more than 15 minutes.  
Remove promptly and rinse thoroughly with water.

**STORAGE:** Place in a cool dry place.

**WARNINGS:** Keep out of reach of children. Use only as directed. Do not put cleaning tablets or cleaning solution in mouth. Do not use to clean natural teeth. Thoroughly rinse appliance in running water before putting in mouth. If ingested, seek medical assistance immediately. In the U.S.A. please call Poison Control Center at 1-800-222-1222.

**CAUTION:** Keep out of reach of children. Do not put cleaning tablet in mouth.

©2007 Raintree Essix, Inc.

Retainer Brite is a registered trademark of Raintree Essix, Inc.



CE

RB-DFU-ENG Rev. 09/2007